



Processor Maintenance Checklist

Daily:

- At the start of the day, check the chemistry levels in the solution trays. Add Six (6) ounces of fresh developer and fixer to the working solution tanks. *Units with auto replenishment ONLY: Check the replenisher tanks as well.*
- Run a couple of clean up sheets to prevent artifacts on film.
- At the end of each day **be sure to turn power switch OFF**, remove the front access cover to allow for cooling and to prevent condensation.
- If the optional water re-circulator is used, replace the water daily.

Weekly:

- Turn power switch OFF**, remove the transport racks from the developer and fixer trays. Wash the racks thoroughly with warm water, scrubbing with a soft bristle brush to remove any chemical residue. After rinsing thoroughly, drain and stand the transport racks on end and allow to dry overnight.
- Clean washer/dryer rack every other week for average use.
- To resume normal operation, rinse dried racks, check for proper operation and place in solution trays. Replenish solutions, turn power switch ON.

Note: Prolonged use of processor without air-drying the racks can cause excessive swelling of the rubber rollers, which may lead to film slippage or loss and excessive transport wear. **Air-drying the racks is very important.**

Monthly:

Monthly procedure is for average users. Heavy volume users should use Easy Clean™ more frequently.

- Turn power switch OFF**, thoroughly clean the processor racks with DENT-X Easy Clean™ (DENT-X Easy Clean™ part #9992602602).
- Drain solutions from the trays. Remove, rinse and scrub trays.
- Units with auto replenishment ONLY: Rinse replenisher tanks and fill halfway with warm water. Flush pumps with warm water by turning the control box off, waiting ten seconds and turning back on. Repeat several times and let the pumps draw air by removing the hose from the tanks. Leave on until the water is purged from the pump. Discard excess water from the replenisher tanks and then re-install the tanks.*
- Refill the tanks with fresh chemistry.
- Remove the dryer rack assembly and clean in the same manner as the other racks.
- Lubricate the main drive assembly as indicated in the manual.
- To resume operations, turn power switch ON.



An AFP Imaging Company

250 Clearbrook Road
Elmsford, N.Y. 10523
800.225.1702
www.dent-x.com