



250 Clearbrook Road, Suite 240
Elmsford, NY 10523

Phone: 914-592-6100

Web: www.ImageWorksCorporation.com

Pre-Installation Checklist for EVA with EVAsoft/ViewAll

Required.

- Have IT support available for the installation.
- Identify locate and specify the HOST/SERVER computer.
 - Physical location.
 - Windows Computer name or IP address.
 - Operating system.
 - Administrator Account name and password.
- Identify, locate, count and specify all Client workstation computers.
 - Physical location(s).
 - Windows Computer name(s) and/or IP address.
 - Operating system(s).
 - Administrator Account name and password.
- Verify that all the target computers meet the minimum requirements.
- Identify any firewalls and antivirus/malware software used on computers or the local network.
 - Have the administrator passwords available if needed.
- If you will be bridging to Practice Management Software, identify and have the PM SW supplier's contact information available.
- If Connecting EVAsoft/ViewAll to other manufactures' hardware (e.g. intra-oral video cameras, TWAIN compatible panoramic x-ray scanners, etc.), please have the other manufacturers' contact information available.
- Internet access to the computers (for registration and if necessary remote diagnostics).

Recommended.

- An extra USB powered hub.
- An extra USB Cable.

A typical installation will take 15 to 20 minutes per computer.

Installation Record (copy for more computers if necessary and file)

Practice Name:	Contact Name:
Main Phone Number:	Email:
Address:	

Computer:	<input type="checkbox"/> Server	<input type="checkbox"/> Client	<input type="checkbox"/> Standalone
Name:	IP Address:		
Operation System	Service Pack:		
Physical Location:			
EVAsoft/ViewAll Installed Version	Date:		
ImageWorks Hardware Connected	<input type="checkbox"/> None <input type="checkbox"/> EVA <input type="checkbox"/> Rotograph D <input type="checkbox"/> Strato D	Serial Number:	
Other Dental Image HW Connected:	Specify Type, Make and Model:		

Computer:	<input type="checkbox"/> Server	<input type="checkbox"/> Client	<input type="checkbox"/> Standalone
Name:	IP Address:		
Operation System	Service Pack:		
Physical Location:			
EVAsoft/ViewAll Installed Version	Date:		
ImageWorks Hardware Connected	<input type="checkbox"/> None <input type="checkbox"/> EVA <input type="checkbox"/> Rotograph D <input type="checkbox"/> Strato D	Serial Number:	
Other HW Connected:	Specify Type and Model:		

Computer:	<input type="checkbox"/> Server	<input type="checkbox"/> Client	<input type="checkbox"/> Standalone
Name:	IP Address:		
Operation System	Service Pack:		
Physical Location:			
EVAsoft/ViewAll Installed Version	Date:		
ImageWorks Hardware Connected	<input type="checkbox"/> None <input type="checkbox"/> EVA <input type="checkbox"/> Rotograph D <input type="checkbox"/> Strato D	Serial Number:	
Other Dental Image HW Connected:	Specify Type, Make and Model:		