



## Registering a Demonstration Version of EVAsoft

- When opening EVAsoft and it hasn't been registered yet you will be prompted to register your version of the software.
- If you do not see that prompt and EVAsoft is open you can click on Help, then License Registration.

**STEP 1:** Enter **DEALERCODE** into the CD Serial Number field

**STEP 2:** Single click **Create Challenge Code**

The screenshot shows a 'License Registration' dialog box with the following content:

**License Registration**

Step 1: Enter the CD Serial Number and Click 'Create Challenge Code' button.

CD Serial Number:

Step 2: Click the 'Activate Computer' button to obtain the 'Activation Code'.

Challenge Code:

Step 3: Enter the Activation Code and Click 'Finish Registration' button to register the software on this computer.

Activation Code:

Red arrows point from the step instructions to the corresponding fields and buttons in the dialog box.

**STEP 3:** Enter **DEALERCODE** into the Activation Code field

**STEP 4:** Single click **Finish Registration**