




250 Clearbrook Road, Elmsford, NY 10523 ♦ (914) 592-6665 ♦ Fax (914) 592-6148 ♦ www.dent-x.com ♦ e-mail: dent-x@dent-x.com

## INSTRUCTION FOR DISABLING THE 'READY' BUTTON:

1. Turn On the machine by pressing the main switch (I/O) on the underside of the timer and wait for the display (number 18) to show the software version (example: 4.02). Once it shows the software version, simultaneously press and hold the 'increase' and 'decrease' keys (numbers 16 and 17, respectively) for 5 seconds.
2. The word 'Pr0' will appear on the display (number 18) for two seconds, and then would be replaced with 'P01'.
3. Scroll from 'P01' to 'P17' by pressing the Exposure Enabling key (number 3).
4. Once you are at 'P17' press the 'Increase' key (number 16). The display will show 'A=1', which means that the 'Ready' button is enabled. Since you want to *disable* the 'Ready' button, press the 'Decrease' key (number 17). The display will now show 'A=0' and the 'Ready' button is disabled.
5. Press the Exposure Enabling 'Ready' key (number 3) to save this configuration. The display now shows 'P18'.
6. Press the Exposure Enabling 'Ready' key (number 3) again to enter User Operation mode.
7. Check that the Green 'Ready for X-ray' LED is constantly illuminated (number 2).

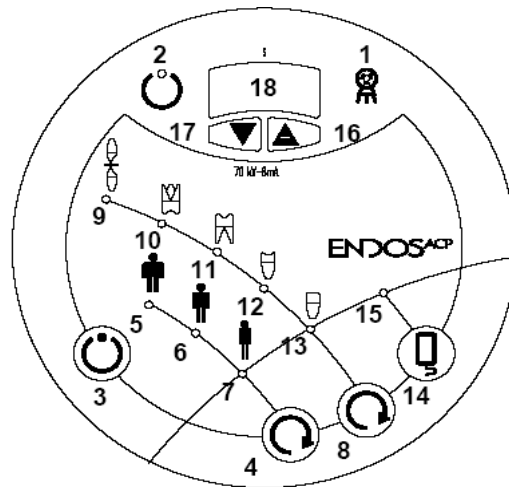

- |                                      |                              |
|--------------------------------------|------------------------------|
| 1. X-ray presence luminous indicator | 10. LED 'Upper Molar'        |
| 2. Green 'Ready for X-ray' LED       | 11. LED 'Lower Molar'        |
| 3. Exposure enabling 'Ready'         | 12. LED 'Premolars'          |
| 4. Selection key 'Size'              | 13. LED 'Incisors / Canines' |
| 5. 'Large Size' LED                  | 14. 'Digital' selection key  |
| 6. 'Medium Size' LED                 | 15. 'Digital' LED            |
| 7. 'Small size' LED                  | 16. Increase key.            |
| 8. 'Tooth anatomic' selection key    | 17. Decrease key.            |
| 9. LED 'Bite-Wing'                   | 18. Three figure display.    |