## ***The U.C.A. platform***

The "U.C.A." (User Customizable Applications) technology allows the user to expand the scope of application of the system in a quick and effective way thanks to upgrade packages that the user will be able to install into the unit by himself by using a normal PC.

The system can be tailored to user's requirements in a unique way and it's open to accept any improvement and change both in user's needs and in examination techniques.

Each upgrade package will contain more than one examination. The following table gives an overview of the packages.

<b>Package</b>	<b>Field of Application</b>
<b>BASE</b>	General dentistry
<b>CEPH</b>	Orthodontics
<b>IMPLANT</b>	Implantology
<b>TMJ</b>	Gnathology, study of the TMJ
<b>SINUS</b>	Implantology, study of the Sinuses
<b>A.D.A. (ADVANCED DENTAL APPLICATIONS)</b>	General dentistry

Part numbers and content of each package are the following:

	<b>U.C.A. packages</b> <i>(Installation performed by the user)</i>
7609504500	<b>IMPLANT</b> <i>(includes Implant 2 and Implant 4 programs, maxillary and mandibular bites, calibrated ruler and DMS software)</i>
7609504600	<b>TMJ</b> <i>(includes open/closed mouth TMJ, bi-axial TMJ and DMS software)</i>
7609504700	<b>Advanced Dental Applications (A.D.A.)</b> <i>(includes improved orthogonality dentition program and DMS software)</i>
7609504800	<b>SINUS</b> <i>(includes P-A maxillary sinus and L-L maxillary sinus and DMS software)</i>
5809316500	DMS software only

The upgrade procedure can be performed by connecting the unit to a PC and following a guided procedure.

The exact sequence of operations is the following:

1. The user realizes that he needs a new upgrade package, let's say the Implant
2. The user places the order to the authorized dealer and **specifies the serial number** of his unit.
3. The dealer orders the UCA packages to Villa, once again specifying the unit's serial number.
4. Villa delivers the package to the dealer. The contents of the package will be:
  - a CD with the upgrade
  - accessories, if required, such as the Implant bites
  - a sealed letter containing the "Activation Code" to be entered into the DMS software during the upgrade procedure.
5. The dealer brings or delivers the package to the end user.
6. The user installs the CD into his PC (connected to Strato 2000) and follows the instructions within the "UCA upgrade" function of the DMS program
7. The user types the activation code when requested
8. The new application package is uploaded into Strato 2000 and the new examination programs are activated

*Note:*

The activation code is unique for each unit and it is strictly related to "Unique Identification Code" (UIC), which is permanently stored inside Strato 2000 and is different for each unit. This means that if you try to install the package into another unit having a different UIC, the system will not work.

That's why it is fundamental to communicate to Villa the equipment's serial number. Without it, we can't deliver the upgrade package.

If the user doesn't have a PC or he doesn't feel comfortable with this kind of operation, than the dealer has to take care of the upgrade procedure.

A laptop PC is required to perform the operation.

The serial connection cable is supplied with the basic Strato 2000 unit and should be available at the doctor's office.

The procedure can be done in fifteen to thirty minutes and requires no hardware operation on the unit.

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*This upgrade package will allow you to further expand the diagnostic target of your STRATO 2000 with new, powerful applications. To install the upgrade package follow the instructions described in the "Quick Start Guide" and, when requested, enter the following activation keys:*

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Should you require further assistance during the installation process, please contact your local Dealer, or the Villa Sistemi Medicali Technical Service at: [service\\_support@villasm.com](mailto:service_support@villasm.com).

Example of Activation letter