


EVAsoft


Web Registration Instructions.

Internet access required.


1. Open EVAsoft the License Registration Prompt will popup. (also accessible through the menu Help-Registration).

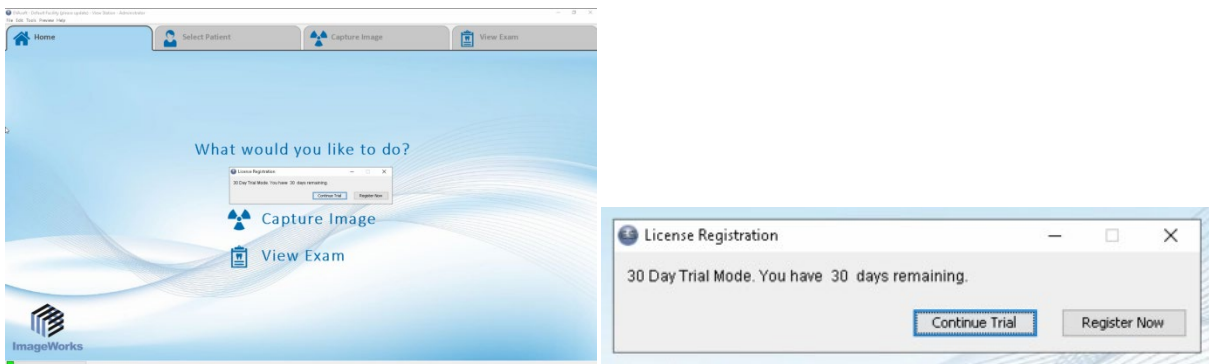



2. Click on Register Now, or click on Help → License Registration.

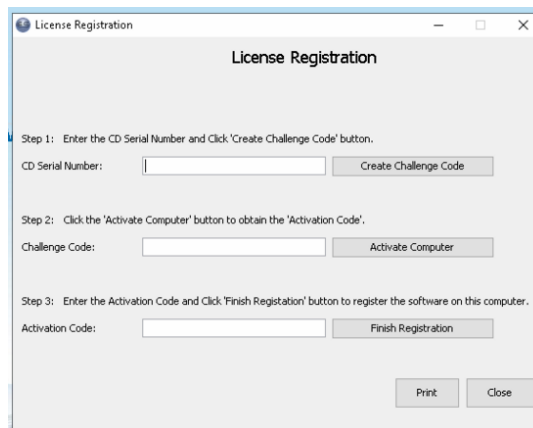


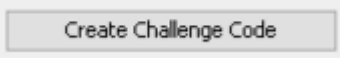

- Carefully enter the CD Serial number provided on their CD case and invoice in the CD Serial Number Text Field and then click the  button. A number will be generated and appear in the Challenge Code Window. This is the challenge code for this computer.

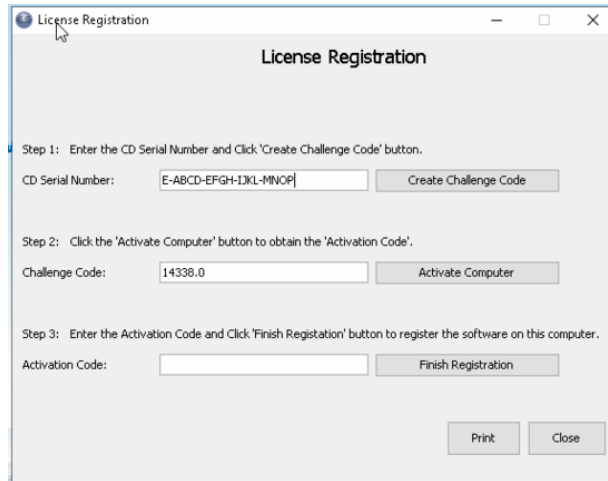

- Now click the  button. The web browser will open to the registration page.

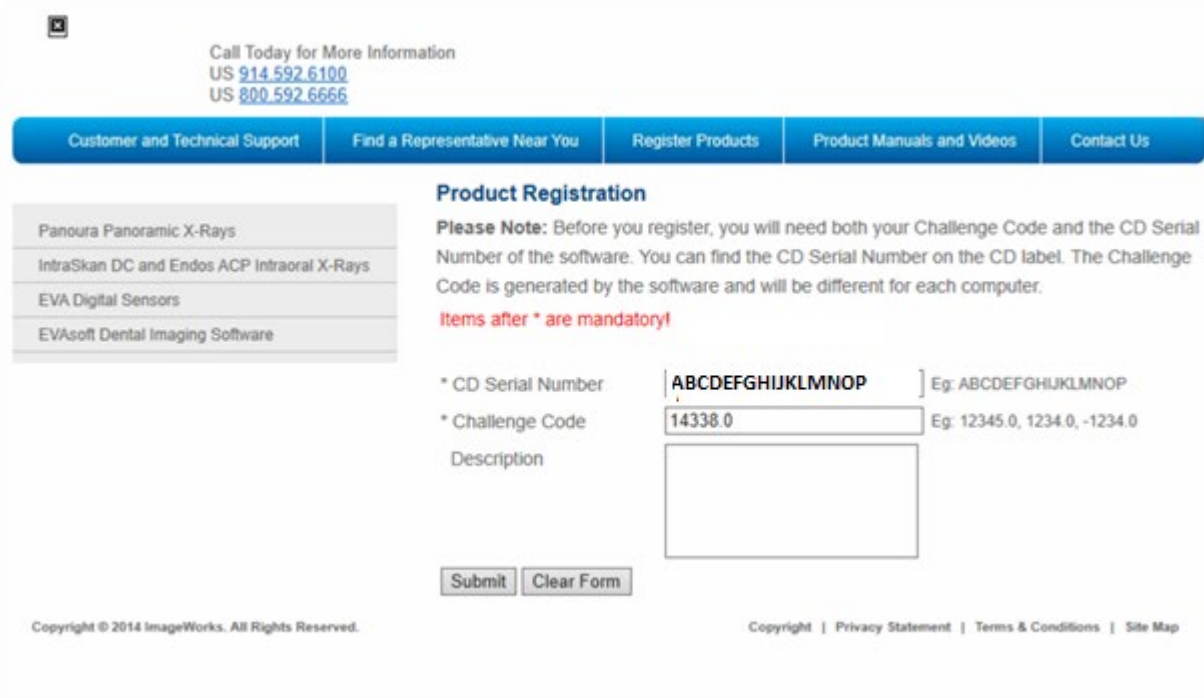

-

- Click the  button.



Submit

7. Enter your information and click



Call Today for More Information
US [914.592.6100](tel:9145926100)
US [800.592.6666](tel:8005926666)

- Customer and Technical Support
- Find a Representative Near You
- Register Products
- Product Manuals and Videos
- Contact Us

- Panoura Panoramic X-Rays
- IntraSkan DC and Endos ACP Intraoral X-Rays
- EVA Digital Sensors
- EVAsoft Dental Imaging Software

Product Registration

Entries after * are mandatory.

* Company Name

* Address

* City

* Country

* State/Province

* Postal/Zip Code

Email Address



* Phone Number


Fax Number


Submit Clear Form



8. The web page will generate and display an Activation code for your computer.

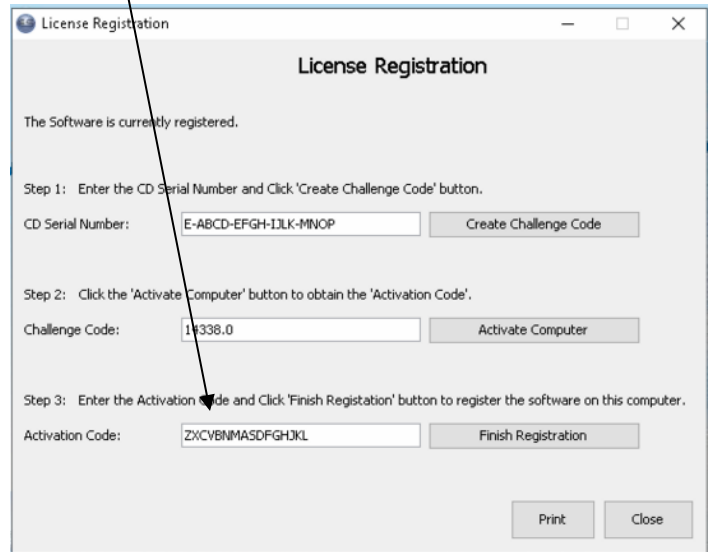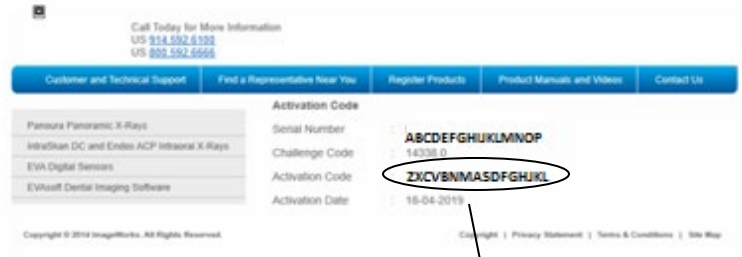

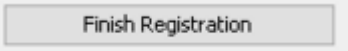9. Enter the activation code displayed on the web page into the Activation code field on the EVAsoft license Registration popup on your local computer and click 

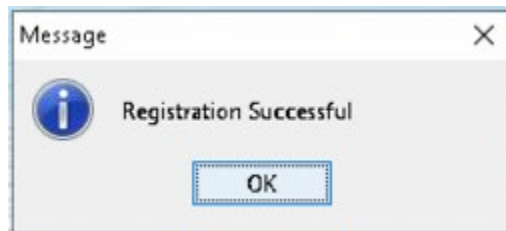

REGISTRATION COMPLETE